



HemoWeld - T A B L E


HEAVY-DUTY SEALING SOLUTION



KEY FEATURES

HemoWeld-Table is the reliable way to perform intensive sterile sealing procedures.

6

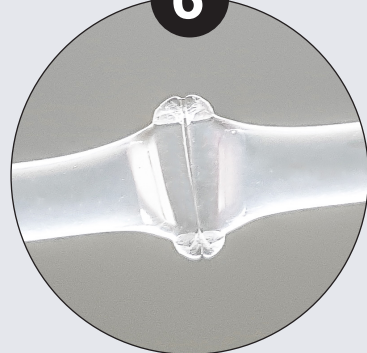

1. **Heavy-duty** - high working capacity
2. **Automatic clamp** - seals in about 1.2 seconds
3. **Self-adjusting RF energy** - to optimize the sealing process
4. **Clamp cap protection** - easy to remove and to clean
5. **Easy snap-apart separation** - a clear-cut notch line makes for an easy separation of the tube
6. **Anti-spark & overheat protection systems** - to avoid damage to system and tubing
7. **Ergonomic handle** - to improve flexibility (optional)
8. **Alerts notifications** - a beep and LED signals to indicate possible problems

7



HemoWeld-Table is a sealer specifically designed for bench use and for **heavy-duty** jobs.

HemoWeld-Table can be used to seal most of standard PVC tubing available on the market: easy to use and to clean, it is characterized by a **high speed** and working capacity. Its sealing clamp automatically closes once the tube is inserted: the seal is done in just about 1.2 seconds.

The protection cap of the clamp is easy to remove and easy to wash, ensuring a better safety for the operator and for the patient. Each seal appears with a central **notch line** to facilitate the snap-apart separation of the two parts; the sterility of the internal blood components is maintained as well. An anti-spark and overheat **protection** systems to avoid damage to the system and tubing it is included as standard. A beep and LED signals indicate the presence of sparks while sealing, of dirty electrodes or of system overheating.

A separate **ergonomic** handle is optionally available to improve the flexibility of work.

Acoustic Alerts

Together with a dynamic LED signal, a beep indicates any problem, such as sparks during the sealing process, dirty electrodes and overheating.

Error prevention

Each seal is wide and strong, with a clear notch line facilitating the snap-apart separation of the 2 parts. A smartly designed clamp cap helps tube positioning and protects the sealing area, and is easy to remove with a quick gesture.

Optimized power consumption

The self adjusting, built-in RF electronics optimize the power consumption, with an eye towards the environment. That ensures a high number of sealing before any overheating, for a continuous heavy usage.

Sales Codes

HWELD-T.DB.00 Bench sealer for blood bags tubing (with plug for auxiliary corded handle)

Regulation

CE marking: Complies with Class 1 Medical Device according to directive 93/42 MDD and 2007/47 MDD

HEMOWELD - T

IS PART OF THE HEMOWELD LINE

A range of products specifically designed to produce sterile seals on standard PVC tubing normally used in conjunction with blood bags.

Discover all the HemoWeld Line on our website!



REV.05 - 2019

For ordering and technical information, please download the full DataSheet from our website.

DELCON srl
Via Matteucci, 25/2720862
Arcore (MB) - Italy
Phone +39 039.617670

